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1. Characteristics

TRAVEL TIM, TRAVEL SNAIL

The pillow is made of shape memory foam to ensure proper support and correct position of the cervical spine and head during travelling or sitting. The pillow is equipped with high-quality, skin-friendly cover with a zipper and adjustable front fastening (velcro in TRAVEL SNAIL) for convenience of usage.

UNICORN TIM

The pillow is made of shape memory foam to ensure proper support and correct position of the cervical spine and head during travelling or sitting. The pillow is equipped with high-quality, skin-friendly cover with a zipper.

BONE TIM

The supportive pillow is made of shape memory foam, so it easily adapts to body, ensuring a proper position of the cervical spine and improving comfort while sitting in a car. The pillow is equipped with high-quality, skin-friendly cover with a zipper.

2. Application

TRAVEL TIM, TRAVEL SNAIL, UNICORN TIM

The pillow is designed for people who spend a lot of time sitting and travelling. It provides support for the head and neck while sleeping or sitting. Anatomically shaped, it provides a comfortable fit and relieves the spine. The use of the pillow has a positive effect on problems resulting from incorrect position of the neck and head during sleep (back pain, muscle tension, tingling and numbness in the limbs, etc.). Recommended prophylactically.

BONE-TIM

The supportive cushion relieves muscle tension in the neck area. It is designed for

people who need stable support for the head and cervical section of the spine while sitting in a car. The use of the pillow has a positive effect on problems resulting from an incorrect position of the neck and head during sitting (back pain, muscle tension, tingling and numbness in the limbs, etc.). Recommended prophylactically.

3. Contraindications

Users with serious medical conditions or changes of the cervical spine should consult a physician or physiotherapist prior to first use of the product.

4. Notes

Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.

5. Technical data

Model	TRAVEL TIM	TRAVEL SNAIL	UNICORN TIM	BONE TIM
Size	29 x 27 x 15 cm	25 x 22 x 13 cm	33 x 28 x 10 cm	29 x 20 x 8 cm
Density	40 kg/m ³	40 kg/m ³	40 kg/m ³	40 kg/m ³
Composition parts	Pillowcase: Polyester 95% Elastane 5% Filling: Thermoelastic foam	Pillowcase: Polyester 95% Elastane 5% Filling: Thermoelastic foam	Pillowcase: Polyester 100% Filling: Thermoelastic foam	Pillowcase: Polyester 95% Elastane 5% Filling: Thermoelastic foam
Pillowcase	zipper, front fastener	zipper, velcro	zipper	zipper
Colour	blue, grey	grey	white	grey

6. Maintenance

- Wash the pillowcase by hand in lukewarm water, do not spin, do not chlorinate, do not iron, do not bleach, do not tumble dry, do not dry clean.
- Dry at room temperature away from intense heat sources.

Please note! Maintenance information can be found on the fabric label.

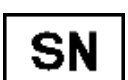
7. Storage and transport conditions

Do not expose the product to direct sunlight, water and moisture.

8. Warranty information

All products distributed by our company are under a warranty, the terms of which are described in the warranty card received at the time of purchase. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

9. Labels

	Reference number		Manufacturer
	LOT number		UDI code
	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors. Learn more at: **www.timago.com**. Thank you for choosing Timago!

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