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**INSTRUCTIONS FOR USE  
MONORAIL OA KNEE BRACE "MONOMAX" O-1260**

**CAUTION!** Before using the device, please consult the instructions for use.

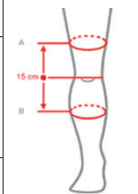
### I. DEVICE DESCRIPTION

The "MONOMAX" O-1260 monorail OA knee brace consists of a body made of modern plastics, a metal connector, straps with an innovative quick fixation system, softening liners with a hook and loop fastening system the position of which can be individually adjusted. The unique design of the hinges allows you to regulate the angle of valgus and varus of the knee depending on your needs, and thus helps to relieve pain and change the pressure in the knee joint.

### II. FEATURES


The brace, thanks to the hinges that allow for control of valgus and varus knee, reduces pressure on disease-affected areas of the knee joint. Pressure changes in the joint are the effect of correcting the axis of the knee joint and improving its functioning. Thanks to the fact that the connector is equipped with a clock, the mobility of knee flexion and extension can be limited, which is useful in the rehabilitation process. Plastic elements are lightweight and durable. The unique quick strap fastening system increases the comfort of putting on and taking off the brace.

### SIZE CHART

Size chart:			
Size	(A) Thigh circumference 15 cm above the center of the patella	(B) Calf circumference 15 cm below the center of the patella	
Universal	39-67 cm	30,5-50,5 cm	

Available in a left or right leg version.

### III. APPLICATION INSTRUCTIONS

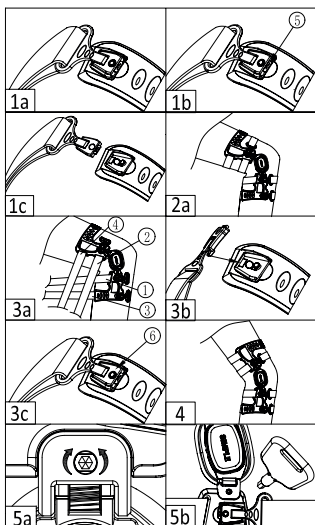- Make sure that the size of the device is properly selected.
- Loosen all straps, unfasten the buckles by pressing and holding button 6.(1a-1c)
- Sit down, put on the brace, the knee should be slightly bent, the center of the clock should align with the center of the patella. (2a)
- Fasten the straps in the correct order; the clock should be supported with the other hand while fastening. The order of fastening the straps: 1. Upper calf strap; 2. Lower thigh strap; 3. Lower calf strap; 4. Upper thigh strap. (3a-3c)
- Adjust the tightness of the straps for personal comfort; straps fastened too tightly will cause problems with blood flow, not sufficient tightness will cause the brace to move on the leg.
- When looking at the brace from the side, the clock may be slightly off the center of the knee joint, which is desired so that it can function without conflict with the knee. (4)
- The valgus/varus adjustment can be done using an Allen key, slowly, taking a few steps after each change of the position. (5a-5b)
- The angle of flexion and extension can be adjusted using the included wedges.
- The length of the straps is adjustable. The strap is to be wrapped and connected with a double-sided hook and loop fastener, the right length is to be set and the excess strap can be cut off with scissors.

### IV. COMPOSITION:

- Nylon 52%
- Poliuretan 27%
- Poliester 14%
- Inne włókna 7%

### V. TERMS OF USE

- The device may only be used as recommended by a physician,
- The method of putting on the brace should be demonstrated by qualified medical personnel, in the presence of the person taking care of the user (if applicable),
- The device and its components should be used only for their intended purpose,
- If pain, swelling, sensory changes or any unusual reactions occur during the use of the device, stop using it immediately and consult your physician,
- Use of the device at night is only permissible if recommended by the attending physician,
- Using the brace for a long period of time may result in limitation of active and passive movements in the joints and muscle atrophy, therefore it is recommended to systematically perform exercises selected by qualified medical personnel as recommended by the attending physician,
- The device should be kept clean following the recommendations given in these instructions for use and on the label placed on the device. Improper cleaning and maintenance of the device may lead to skin lesions,
- The device cannot be used if any part is missing. Two of the same or different devices cannot be used at the same time.

In case of any serious incident that has occurred in relation to the device, please report to the manufacturer and the competent authority of the Member State in which the user or patient is established.

### VI. INDICATIONS

The device can be used for:

- Unicompartmental, lateral or medial gonarthrosis of the knee joint
- Postoperative treatment of cartilage or meniscus

### VII. CONTRAINDICATIONS

- Existing problems with lymph drainage,
- A too tight brace may lead to increased pain, while a too loose brace will not provide the intended effect,
- In the event of skin changes in the areas where the brace adheres, its use is limited and possible only under medical supervision,
- It is necessary for the attending physician to assess the use of the brace in people with memory disorders of various origins, people with mental disorders and children.

### VIII. CLEANING AND MAINTENANCE

It is recommended to wash the device regularly. Before washing, fasten all hook and loop fasteners of the brace.

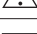
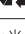
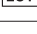
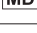
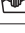
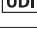
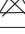
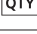
- Hand wash in water with a neutral, anti-allergic detergent, at 30 degrees,
- Rinse in lukewarm water, do not use fabric softeners,
- Do not bleach,
- Do not iron,
- Do not dry clean,
- Do not tumble dry,
- Dry flat in a shaded and airy place - do not expose the device to direct sunlight.

### IX. WARRANTY

The manufacturer assures the customer that the item sold is brand new and of high quality.

The device is covered by a warranty (detailed warranty conditions can be found on the Manufacturer's website).

### X. SYMBOL DESCRIPTION

	Manufacturer		This way up
	CE marking		Keep dry
	Catalogue number		Fragile, handle with care
	Date of manufacture		Do not cut
	Consult instructions for use		Do not use if package is damaged and consult instructions for use
	Caution		Recyclable packaging
	Batch code		Keep away from Sunlight
	Medical device		Hand wash
	Unique Device Identifier		Do not bleach
	Quantity		Do not iron
	Model number		Do not dry clean
	Date of issue		Do not tumble dry

**REHA FUND Sp. z o.o.**

ul. Staniewicka 14,  
03 - 310 Warszawa, Polska (Poland)  
e-mail: [info@rehafund.pl](mailto:info@rehafund.pl)  
[www.rehafund.pl](http://www.rehafund.pl)

Date of issue of the instructions for use: 09.2024  
I/O-1260/1/05.09.2024