INSTRUKCJA OBSŁUGI
UŽIVATELSKÝ MANUÁL
BRUGSANVISNING
BENUTZERHANDBUCH
ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ
USER MANUAL
KASUTUSJUHEND
KORSNIČKE UPUTE
FELHASZNÁLÓI ÚTMUTATÓ
MANUALE D'USO
NAUDOJIMO INSTRUKCIJA
LIETOTĀJA INSTRUKCIJA
MANUAL DO UTILIZADOR
MANUAL DE INSTRUCCIUNI
NÁVOD NA POUŽITIE
NAVODILA ZA UPORABO
BRUKSANVISNING

PL INSTRUKCJA OBSŁUGI	4
CS UŽIVATELSKÝ MANUÁL.....	8
DA BRUGERVEJLEDNING.....	11
DE BENUTZERHANDBUCH	14
EL ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΤΗ	18
EN USER MANUAL	22
ET KASUTUSJUHEND	26
HR PRIRUČNIK ZA UPOTREBU	30
HU FELHASZNÁLÓI ÚTMUTATÓ.....	34
IT MANUALE D'USO	38
LT NAUDOJIMO INSTRUKCIJA.....	42
LV LIETOTĀJA INSTRUKCIJA	46
PT MANUAL DO UTILIZADOR.....	50
RO MANUAL DE INSTRUCȚIUNI	54
SK NÁVOD NA POUŽITIE	58
SL NAVODILA ZA UPORABO	62
SV BRUKSANVISNING	65

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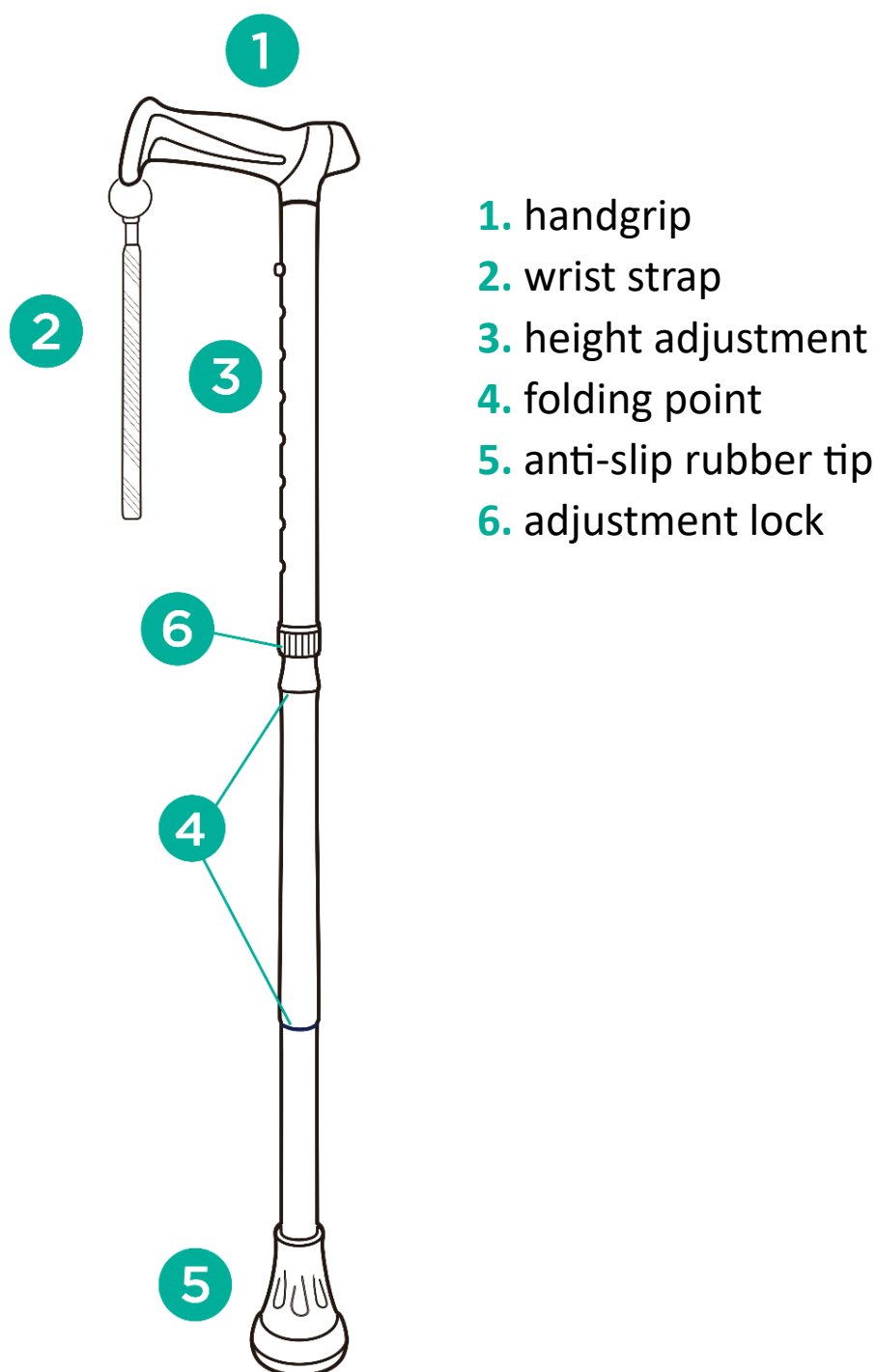
Product code: PrimeTim

1. Characteristics

The foldable cane consists of ergonomic handgrip, high-quality aluminium shaft and anti-slip rubber tip. Thanks to height adjustment and the possibility of folding the cane into three parts, it is very convenient for daily use.

2. Application

The cane is ancillary equipment which is used to increase the area of support in case of patients with reduced mobility. It is also recommended to patients who need help and support while walking and it can also be used as an aid in keeping balance after the injuries of lower limbs or trunk.

Contraindications: coordination and balance disorders, advanced rheumatic changes, injuries and dysfunctions of upper limbs which may hinder the user to support on the cane.

3. Adjustment

- To adjust the cane, unscrew the adjustment lock and press the locking pin. Select appropriate length and lock pin, tightening adjustment lock afterwards. The cane is adjustable in 3 cm increments.
- To set the proper length of the cane, stand upright and bent upper limb in elbow joint at the angle of 20°.
- The cane should be adjusted individually to the user so that the handgrip is at hip joint level.
- Before use, ensure that the cane is stable and the locking pin fully protrudes through adjustment hole of the cane.

NOTE! In case of doubts concerning the technique of moving with the use of cane, consult a person who has appropriate knowledge or qualifications.

4. Technical data

Cane type	foldable, aluminium
Height adjustment	78-96 cm
Max. capacity	159 kg
Weight	0,50 kg
Colour	pink, blue

5. Notes

Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.

6. Cleaning and maintenance

- Clean the cane using only water and mild detergents, dry thoroughly.
- **Do not use:** strong detergents, abrasive agents, bleach, petrol nor solvents (it may cause frame damage, visible defects, and in consequence – loss of warranty rights).
- It is recommended to check rubber tip regularly. Replace worn tip immediately.

7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date
	Serial number		Medical device
	Note		Please read the instruction

The manufacturer checked for compliance with essential medical devices requirements.

Our complete line of products is available from our distributors.

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